


Vial[^]PRO Instructions

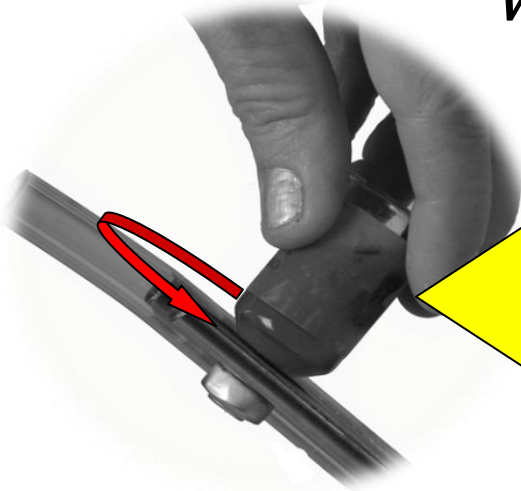


- Turn medication vials upside down then twist and turn while inserting bottles neck between the rods on Vial[^]PRO.

- If needed place the syringe cap firmly into the end of the hose extending from Vial[^]PRO.
- Twist and pull to remove syringe from the cap leaving the cap behind.




- Now with both of your hands are in control of any syringe.
- Pull dosage and administer.
- **Dispose of syringe properly do not recap if at all possible.**

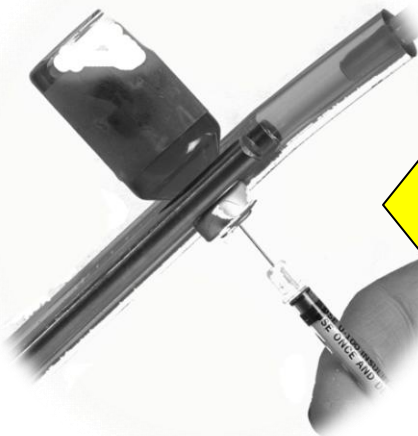

• **Only if necessary!**

- Return syringe to the cap.

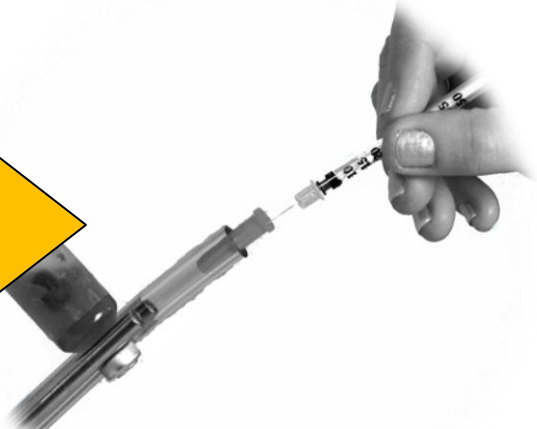

- Grasp the syringe cap and pull from Vial[^]PRO and transport safely.

